

# Status ASSURE

Accurate Results in **5 Minutes**

## Drugs of Abuse Test

- Simple, Fast & Reliable • Easy to Read • FDA Approved



Collection Cup with Test Lid

1. Secure lid test device to the filled specimen cup.
2. Place the cup on its side to activate testing.
3. Read results in 5 minutes.
4. Compare the adulteration results to the color chart card provided.



Drug Screen Card

1. Remove the cap to expose the sampling tips.
2. Dip into urine specimen, replace cap and place device on a flat surface.
3. Read results in 5 minutes.
4. Compare the adulteration results to the color chart card provided.

ADULTERATION COLOR COMPARISON CHART			
TEST AND READING TIME	ABNORMAL (LOW)	NORMAL	ABNORMAL (HIGH)
<b>Creatinine</b> 45 seconds	Negative	20 50 100 200mg/dl	
<b>Nitrite</b> 45 seconds		0 0.1-0.2 0.5-5.0	>15mg/dl
<b>Glutaraldehyde</b> 45 seconds		Negative	Positive
<b>pH</b> Immediate	2 3	4 5 7 9	<10
<b>Specific Gravity</b> 45 seconds	1.00	1.005 1.015 1.025	>1.030
<b>Oxidants</b> 60 seconds	Negative		Positive

Adulteration Color Chart

Read the results of the adulteration test by visually comparing the color of reagent pads to the color chart at the time indicated.

### ORDERING INFORMATION

#### CUPS

6 Panel Cup  
MET/AMP/COC/OPI-2000/THC/PCP\*

11 Panel Cup  
THC/OPI-300/OXY/MDMA/TCA/MET/AMP/BZO/BAR/COC/PCP\*

11 Panel Cup  
THC/OPI-2000/OXY/MDMA/TCA/MET/AMP/BZO/BAR/COC/PCP\*

#### DRUG SCREEN CARDS

Status Assure 6 Panel Drug Screen Card  
MET/AMP/COC/OPI-2000/THC/PCP\*

Status Assure 11 Panel Drug Screen Card  
THC/OPI-300/OXY/MDMA/TCA/MET/AMP/BZO/BAR/COC/PCP\*

Status Assure 11 Panel Drug Screen Card  
THC/OPI-2000/OXY/MDMA/TCA/MET/AMP/BZO/BAR/COC/PCP\*

\*All cups and cards include an adulterant strip with Cr/Ni/pH/Bi/S.G.

#### KIT SIZE

25 Cups/Box

25 Cups/Box

25 Cups/Box

25 Cards/Box

25 Cards/Box

25 Cards/Box

#### ITEM #

76025AA2K

71125AA

71125AA2K

16025AA2K

11125AA

11125AA2K

lifeSign

A PBM Group Company

71 Veronica Avenue  
Somerset, New Jersey 08873  
800.526.2125 732.246.3366

Fax: 732.246.0570

[www.lifesignmed.com](http://www.lifesignmed.com)

lifeSign is a registered trademark of Princeton BioMeditech Corporation. Printed in USA. All rights reserved.