


# Instructions for the rapid **Status**™ COVID-19 Antigen Test For Home Use

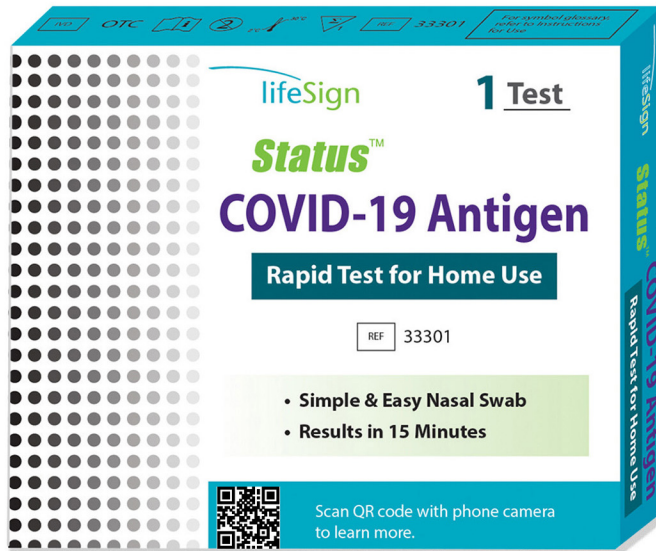

# Have these items ready before starting a test \*



1 Test in  
foil pouch



1 Swab  
in wrapper



1 Reagent  
Capsule



Clock or  
Timer

\* Please read the complete package insert for details

# STEP 1

Open the pouch  
and remove the test



Place the test on  
a flat surface



## STEP 2

Twist the tab  
to remove top




Squeeze the capsule to empty  
all of the Reagent into the  
Extraction Well



## STEP 3



Remove the swab from the wrapper

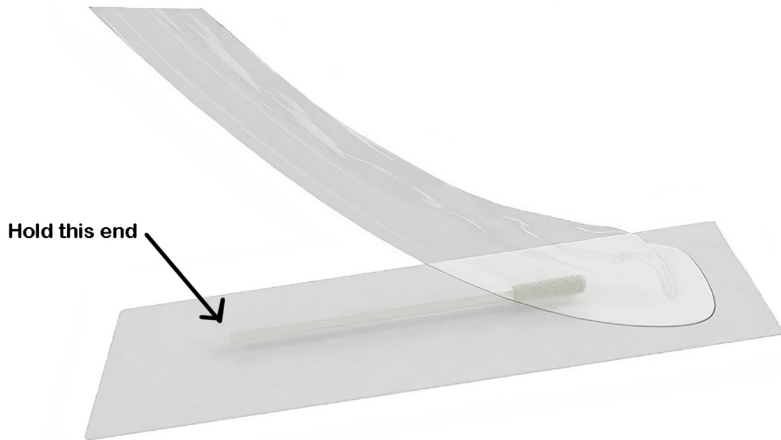

**OPEN THE SWAB WRAPPER AT THE STICK END. BE CAREFUL  
NOT TO TOUCH THE FOAM TIP OF THE SWAB.**

## STEP 4

Insert the swab into the nose and rotate the swab 5 times while pressing against the nostril. Using the same swab, repeat on the other side.

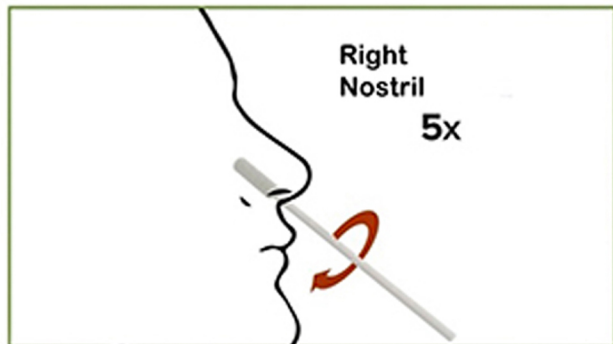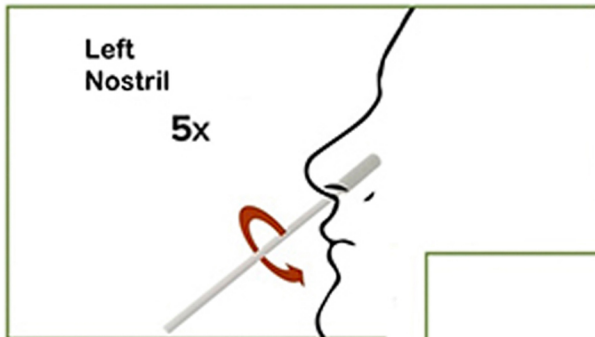

# STEP 5

Insert the swab into the  
swab stand  
Rotate the swab 3x



Leave the swab in the device  
and set a timer for 1 minute



# STEP 6

After 1 minute, rotate the swab again 3x.  
Remove swab and discard.





## STEP 7

Gently raise the device upright and let stand for 3 seconds



Slowly lay the test back down and set a timer for 15 minutes



## STEP 8

# Read the Results



**NEGATIVE  
COVID-19**



**POSITIVE  
COVID-19**



**INVALID**

Repeat test  
using a new device